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Stand der Information: November 2007

INSTRUCTIONS FOR USE

VISLUBE® – Sodium hyaluronate 0.18 %.
For the lubrication of contact lenses during wear.
Sterile, free from preservatives.

Composition:
Each 1 ml of hypotonic solution contains 1.8 mg sodium hyaluronate, sodium chloride, potassium chloride, disodium-hydrogenphosphate, sodium citrate, magnesium chloride, calcium chloride and water for injection.

Indications:
Sensation of dryness, discomfort or distorted vision during contact lens wear.

Contra-Indications:
Hypersensitivity to the product.

Interactions:
VISLUBE® may modify the effects of drugs administered concomitantly on the eye.

Dosage and Administration:
Unless otherwise directed by your contact lens specialist, instill 1–2 drops of VISLUBE® four times a day on the surface of the contact lens during wear or in the conjunctival sac. After blinking the solution will disperse and form a transparent and longlasting coating of the surfaces of the eye and lens.

Warnings:
Do not touch the tip of the opened container and do not place the tip in direct contact with the eye. Do not use VISLUBE® if the container is damaged. As VISLUBE® does not contain preservatives, any solution not used immediately after opening should be discarded.

Store between 2 °C and 25 °C! Do not use VISLUBE® after the expiry date.

Keep out of the reach of children!

Characteristics and mode of action:
VISLUBE® contains sodium hyaluronate, a natural polymer, which is also present in structures of the human eye. The special physical characteristics of sodium hyaluronate means VISLUBE® has important “viscoelastic” properties. VISLUBE® offers a high resistance to the clearance process induced by blinking, thus providing a stable coating on the surface of the eye and lens.

Therefore VISLUBE® combines long lasting relief with maximum comfort. VISLUBE® is free from preservatives and has excellent tolerability.

Presentation:
20 Monodose units of 0.3 ml.
60 Monodose units of 0.3 ml.

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